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Chargennummer

Type of preparation

Ihdent® Dentatrend® Orange Solvent skin-friendly cleaning preparation for the thorough removal of patch and adhesive remnants, plaster, zinc, eugenol pastes, alginate and cement remnants on the skin, paint and plastic-coated (polypropylene, polyethylene) surfaces. With its natural orange oils, Ihdent® Dentatrend® Orange Solvent cleans gently and prevents the skin from drying out excessively. Ihdent® Dentatrend® Orange Solvent is economical to use.

Areas of use

Ihdent® Dentatrend® Orange Solvent cleaning preparation for the thorough removal of remnants such as:

- Patch and adhesive remnants
- Plaster, zinc, eugenol pastes, alginate and cement remnants on the skin, paint, and hard-plastic-coated surfaces and metals.

Application

Put a few drops onto a cloth or cotton-wool swab and wipe the areas concerned. After removing any excess using a clean cloth, wipe again if required. A fine orange fragrance remains.

Important: Ihdent® Dentatrend® Orange Solvent should not be used on silicone, rubber or Plexiglas. High concentrations of orange terpenes can cause sensitive surfaces to soften and deform.

Product composition

Pure orange terpenes derived from naturally pressed orange peel.

Chemical and physical details

Appearance of concentrate: clear fluid
Density (20°C): 0.84 (± 0.015)

Precautionary statements

- P210** Keep away from heat / sparks / open flames hot surfaces – no smoking
- P241** Use explosion-proof electrical / ventilating / lighting / equipment.
- P261** Avoid breathing dust / fume / gas / mist / vapours / spray.
- P280** Wear protective gloves / protective clothing / eye protection / face protection.
- P273** Avoid release into the environment.
- P301 / 310** IF SWALLOWED: immediately call a POISON CENTRE or doctor / physician.
- P331** Do NOT induce vomiting.
- P405** Store locked up.
- P501** Dispose of contents / container in compliance with local / regional / national / international regulations.

Warranty

The manufacturer guarantees that its products are of flawless quality. Recommendations made are based on trials and practical experience. However, the manufacturer cannot be held liable for any incorrect use or any damages caused as a result.

Labelling

UN no. 2319

Storage instructions

Do not use after expiry date. Store only in the original container. Do not store together with flammable substances / oxidants. Keep container closed tightly. Store container in a well-ventilated place. Store in a dry, dark and cool place. Protect against heating / overheating.

Special note

Always read the Instructions for Use and the Material Safety Data Sheet before use.

Keep out of reach of children.

Delivery units

500 ml REF D0065

Expert opinions: Available on request.

9999-D0065-03

Revision issued 2015-03

Ihdent® and **Dentatrend®** are registered trademarks.

Dr. Ihde Dental AG

Dorfplatz 11
CH-8737 Gommiswald/SG
Tel. +41 (0)55 293 23 23
Fax +41 (0)55 293 23 00
contact@implant.com
www.implant.com

Distribution by

Dr. Ihde Dental GmbH
Erfurter Str. 19
D-85386 Eching/München
Tel. +49 (0)89 319 761-0
Fax +49 (0)89 319 761-33
info@ihde-dental.de
www.ihde-dental.de
www.desinfektion.com

Danger

H226

Flammable liquid and vapour.

H304

May be fatal if swallowed and enters airways.

H400

Very toxic to aquatic life.

H410

Very toxic to aquatic life with long lasting effects.

H315

Causes skin irritation.

H317

May cause an allergic skin reaction.

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